


Goff-Bol™ Applicator Usage Guide



1 - Loading the Goff-Bol™ Applicator

Slide the pin from the applicator to the outward position.
Insert two boluses.

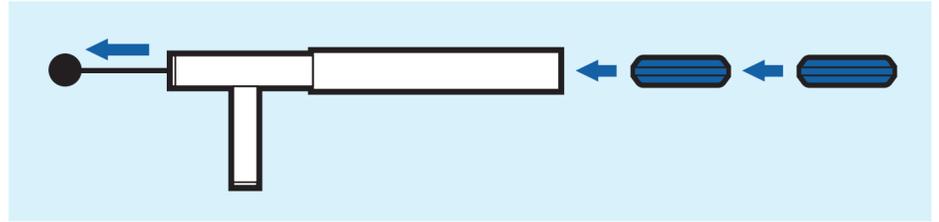

2 - Positioning (figure 1)



If you are right-handed, place yourself on the cow's right side, using your left knee for leverage under the cow's head.




3 - Applicator Insertion (figures 2 & 3)

Grab the cow's upper jaw with your left hand and gently glide the Goff-Bol™ Applicator into the cow's mouth.
Guide the applicator into the cow's mouth until the handle is in the corner of the mouth (over the tongue).

Do not use force - let the cow allow you to insert the applicator.

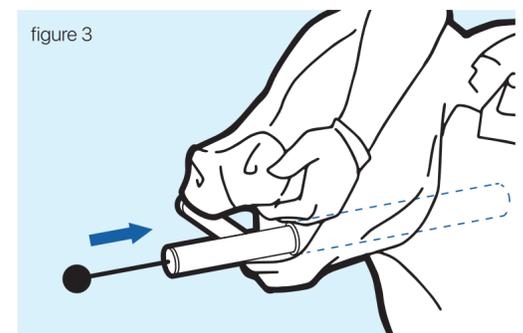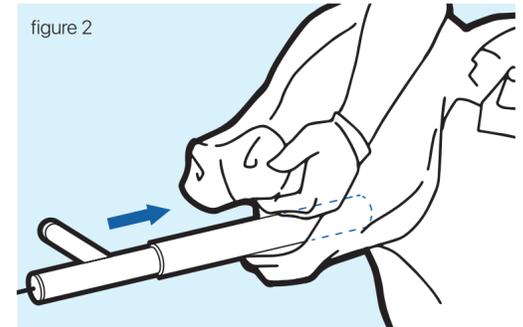


4 - Releasing the First Bolus

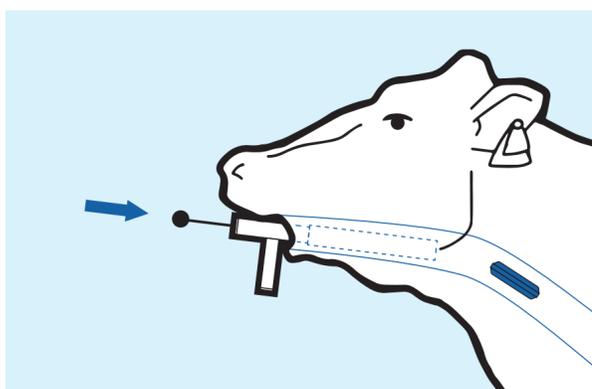


When the applicator is properly inserted, slide the pin inward to release the first bolus. Allow a few seconds to swallow.

5 - Releasing the Second Bolus

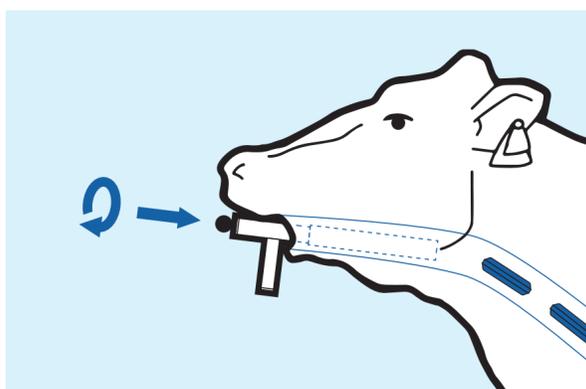

Turn the pin clockwise to align the keyhole and slide pin forward to release second bolus.

6 - Applicator Removal

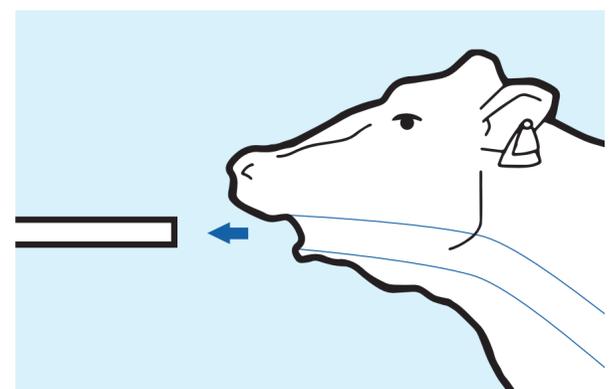

Wait a moment for the cow to swallow and gently remove applicator. Allow access to water.

WARNING

Incorrect use of the applicator may result in the injury and/or death of the animal. Do not use the applicator unless you are appropriately trained. Inspect applicator for wear/damage prior to use. A damaged or worn applicator should not be used. Ensure the applicator is being used on the correct bolus and type of animal being treated.